Shading Liquids

9012374	Zirlux Shading Liquid - Dentine, 0M1, 100ml
9012375	Zirlux Shading Liquid - Dentine, 0M2, 100ml
9012377	Zirlux Shading Liquid - Dentine, 0M3, 100ml
9012380	Zirlux Shading Liquid - Dentine, A1, 100ml
9012381	Zirlux Shading Liquid - Dentine, A2, 100ml
9012382	Zirlux Shading Liquid - Dentine, A3, 100ml
9012383	Zirlux Shading Liquid - Dentine, A3,5, 100ml
9012384	Zirlux Shading Liquid - Dentine, A4, 100ml
9012385	Zirlux Shading Liquid - Dentine, B1, 100ml
9012386	Zirlux Shading Liquid - Dentine, B2, 100ml
9012387	Zirlux Shading Liquid - Dentine, B3, 100ml
9012388	Zirlux Shading Liquid - Dentine, B4, 100ml
9012389	Zirlux Shading Liquid - Dentine, C1, 100ml
9012390	Zirlux Shading Liquid - Dentine, C2, 100ml
9012391	Zirlux Shading Liquid - Dentine, C3, 100ml
9012392	Zirlux Shading Liquid - Dentine, C4, 100ml
9012394	Zirlux Shading Liquid - Dentine, D2, 100ml
9012395	Zirlux Shading Liquid - Dentine, D3, 100ml
9012396	Zirlux Shading Liquid - Dentine, D4, 100ml

9012397	Zirlux Shading Liquid - incisal for painting technique on shades (0M3, A1, A2, B1, B2)
9012398	Zirlux Shading Liquid - incisal for painting technique on shades (A3, A3,5, A4, B3, B4)
9012400	Zirlux Shading Liquid - incisal for painting technique on shades (C1; C2; D2)
9012401	Zirlux Shading Liquid - incisal for painting technique on shades (C3; C4; D3; D4)
9012402	Zirlux Shading Liquid - incisal for painting technique on shades (0M1; 0M2)

9012403	Zirlux Shading Liquid - Effects, light pink, 30ml
9012405	Zirlux Shading Liquid - Effects, dark pink, 30ml
9012406	Zirlux Shading Liquid - Effects, purple, 30ml
9012408	Zirlux Shading Liquid - Effects, yellow, 30ml
9012409	Zirlux Shading Liquid - Effects, orange, 30ml
9012410	Zirlux Shading Liquid - Effects, light brown, 30ml
9012411	Zirlux Shading Liquid - Effects, dark brown, 30ml
9012412	Zirlux Shading Liquid - Effects, blue, 30ml
9012413	Zirlux Shading Liquid - Effects, light grey, 30ml
9012414	Zirlux Shading Liquid - Effects, dark grey, 30ml
9012415	Zirlux Shading Liquid - Effects, graphite, 30ml
9012416	Zirlux Shading Liquid - dentin lightener, 30ml

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Distributed by (in US only)
HENRY SCHEIN INC.
135 DURYEA ROAD
Melville, NY 11747 USA

HENRY SCHEIN SERVICES GmbH
MONZASTRASSE 2A
D-63225 LANGEN, GERMANY
TEL: +49 6103 757 5000
Rev.2017/06

INSTRUCTIONS FOR USE

1. Indications

Zirlux Shading Liquids are liquids for coloring pre-sintered zirconia frameworks.

2. Contra-Indications

The contra-indications of the framework material used apply.

3. Safety instructions

Wear gloves and safety glasses. In the event of contact with the skin or eyes, rinse with water and consult a physician if irritation persists.

4. Handling and storage

This medical device should only be handled by trained personnel. Only process defect-free, dry frameworks zirconia (ZrO_2) milling blanks. The Zirlux Shading Liquids solutions should be stored in the original container, tightly closed, at a maximum of 25°C / 95°F.

5. Processing

Immersion technique:

- Select the coloring liquid according to the framework material and the color to be achieved (shade combination guide can be found in the full instructions at www.zirlux.com)
- Remove any residual dust from the ZrO_2 framework by careful cleaning with compressed air. Zirconia frameworks processed in wet-milling machines must be dried immediately after the wet-milling process
- Work place preparation: Prepare a timer, paper cloths, plastic forceps and clean immersion container plus an applicator
- Shake liquid bottles well before use
- Fill an adequate amount of dentine color into the immersion container
- For monolithic work with an enamel effect: apply the incisal fluid sparingly to the occlusal/incisal area of the ZrO_2 framework twice in succession with the applicator Place the ZrO_2 framework (with/without prior application of incisal fluid) into the immersion container with the dentine color for approximately 20 seconds
- After this time, remove the framework from the liquid using the plastic forceps and carefully dab off any excess liquid with a clean paper cloth
- Place the colored ZrO_2 framework onto a clean, non-absorbent, temperature-resistant surface (e.g. sintering plate) and dry in an oven directly after coloring for around 30 minutes at approximately 90°C / 194°F.
- After drying, sinter the framework in accordance with the instructions for the framework material in question. Please note the separate sintering instructions at www.zirlux.com.

Painting technique: see full instructions at www.zirlux.com.

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EN **Danger: P 280:** Wear protective gloves / eye protection. **P 305 + P 351 + P 338:** IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. **P 403 + P 233:** Store in a well-ventilated place. Keep cool. **P 501:** Dispose of contents/container in accordance with local/regional/national/international regulations. **P302+P352:** IF ON SKIN: Wash with plenty of water. **P 310:** Immediately call a POISON CENTER/doctor. **P261:** Avoid breathing dust/fume/gas/mist/vapours/spray. **H 315:** Causes skin irritation **H318:** Causes serious eye damage. **H335:** May cause respiratory irritation

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